



Remove the battery pack from the main unit and remove the sticker from the contact points.



Reinsert the battery pack and attach the rubber cover to the Avya Steam Inhaler.



Fill the water tank with Avya Aqueous Solution.

4




Press and hold the power button for 3 seconds to power on the Avya Steam Inhaler

5



Adjust the temperature to your comfort and enjoy soothing, germ free steam!

A V Y A QUICK START GUIDE

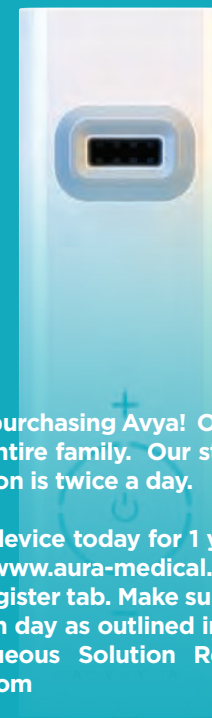

Thank you for purchasing Avya! Our steam inhaler is great for the entire family. Our standard treatment recommendation is twice a day.

Register your device today for 1 year of protection. Please visit: www.aura-medical.com/support and click on the Register tab. Make sure to clean Avya at the end of each day as outlined in the User Manual. For Avya Aqueous Solution Refills please visit aura-medical.com

Note: It is normal for Avya to have a new, factory smell. To eliminate this smell we recommend running a cycle of the following solution through the water tank: 6 Tbs Tap water + 0.5 tsp of baking soda + 0.5 tsp vinegar. Repeat if necessary.